DIAPLUS[®]

C+C DIAMOND INSTRUMENTS

PARTNER AG

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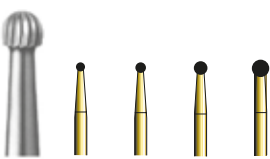
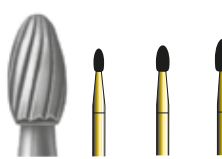
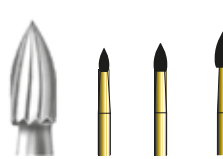
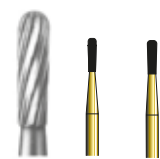
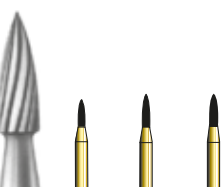
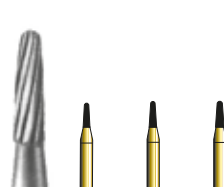
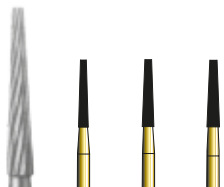
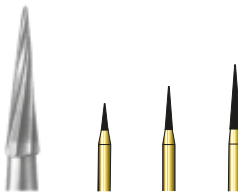
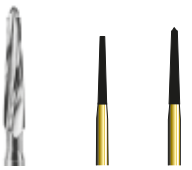
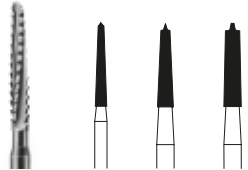
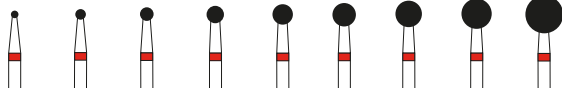
CarDia Instrument for surgical use

141

REF.-No	141	141	141	141	141	141	141	141	141	
Fig.-No ISO	001251	001251	001251	001251	001251	001251	001251	001251	001251	
ISO x 1/10 mm	P	27	27	27	27	29	29	30	32	33
205 RA L		010	014	018	023	027	031	035	040	050
206 RA XL		010	014	018	023	027	031	035	040	050
104 HP		010	014	018	023	027	031	035	040	050

Chirurgische Bohrer 6-12 Schneiden · Surgical burs 6-12 blades · Fresas de cirugía
Fraises chirurgicales 6-12 lames · Frese chirurgiche 6-12 lame

CSTT

REF.-No	CSTT	CSTT
Fig.-No ISO	287484	287484
ISO x 1/10 mm	P	
313 FG S		
314 FG S	34	015 015L
316 FG XL		

Ceramic Soft Tissue Trimmer
for surgical use

Bohrerständler, sterilisierbar · Burs-holder, sterilizable · Fresero, esterilizabile · Support pour fraises, stérilisable · Supporto per frese sterilizzabile

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GB APPLICATIONS

- Removal of crowns and bridges
- Trimming and finishing of acrylics (Composites) and all conventional filling materials
- Burs for excavating
- Surgical burs
- Ceramic Soft Tissue Trimmer for surgical use

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GB USER INFORMATION - HYGIENE RECOMMENDATIONS

User informations

- Use adequate supply of water spray in order not to damage the tooth and contiguous tissues. The water should be distributed over the entire length of the working part. A deflection of the coolant jet can cause heat related damages.
- Additional cooling is required for instruments with a total length exceeding 19 mm and for instruments with an head diameter exceeding 2 mm.
- Immediately remove any instruments that are damaged, bent or do not run concentrically.
- Pressure should be 0.3 - 2N (30 - 200p) for FG instruments and 2 - 5N (200 - 500p) for HP instruments, in other words work with as little pressure as possible.
- The turbine, right angle and handpiece must be in perfect technical conditions.
- Insert the instruments carefully and without using force. Miniature instruments should be inserted no further than the end of the cylindrical part.
- Avoid canting or levering when grinding.
- Observe the recommended speeds indicated in the table below and on packages.
- Full speed should be reached outside the mouth of the patient. Technicians should run the handpiece at full speed before application to the product on which is being performed.

Hygiene recommendations (desinfection, cleaning, sterilization and storage)

- The instruments must always be disinfected, cleaned and sterilized before they are used on patients.
- Immediately place used instruments in a special anti-corrosive disinfectant bath for a short time otherwise their colour code may come off. Do not use too aggressive products (hydrochloric acid, hydrogen peroxide,...), they may corrode the instruments.
- Clean the instruments in an ultrasonic bath or under running water (preferably distilled water).
- After cleaning, immediately dry the instruments otherwise they can corrode.
- Before sterilization, place the instruments in stands and trays suitable for sterilization.
- Sterilize the instruments. Autoclave: 18 min. 134°C, 2 bars.
- After sterilization, keep the instruments in dust-free places. Instruments used for surgery should be kept in sealed containers.

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AZ & PARTNER AG