DIAPLUS[®]

C+C DIAMOND INSTRUMENTS

PARTNER AG

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
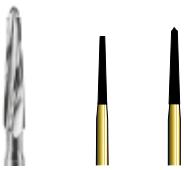
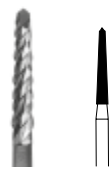
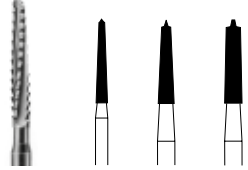
CarDia Instrument for surgical use

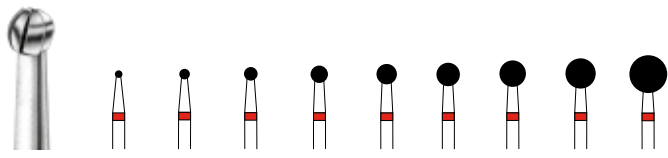141

REF.-No	141	141	141	141	141	141	141	141	141	141
Fig.-No ISO	001251	001251	001251	001251	001251	001251	001251	001251	001251	001251
ISO x 1/10 mm	P	27	27	27	27	29	29	30	32	33
205 RA L	◆	010 014 018 023 027 031 035 040 050								
206 RA XL	◆	010 014 018 023 027 031 035 040 050								
104 HP	◆	010 014 018 023 027 031 035 040 050								

Chirurgische Bohrer 6-12 Schneiden · Surgical burs 6-12 blades · Fresas de cirugía
Fraises chirurgicales 6-12 lames · Frese chirurgiche 6-12 lame

CSST

REF.-No	CSST	CSST
Fig.-No ISO	287484 287484	
ISO x 1/10 mm	P	
313 FG S		
314 FG	34	015 015L
316 FG XL		

Ceramic Soft Tissue Trimmer for surgical use

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GB APPLICATIONS

- Removal of crowns and bridges
- Trimming and finishing of acrylics (Composites) and all conventional filling materials
- Burs for excavating
- Surgical burs
- Ceramic Soft Tissue Trimmer for surgical use

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GB USER INFORMATION - HYGIENE RECOMMENDATIONS**User informations**

- Use adequate supply of water spray in order not to damage the tooth and contiguous tissues. The water should be distributed over the entire length of the working part. A deflection of the coolant jet can cause heat related damages.
- Additional cooling is required for instruments with a total length exceeding 19 mm and for instruments with an head diameter exceeding 2 mm.
- Immediately remove any instruments that are damaged, bent or do not run concentrically.
- Pressure should be 0.3 - 2N (30 - 200p) for FG instruments and 2 - 5N (200 - 500p) for HP instruments, in other words work with as little pressure as possible.
- The turbine, right angle and handpiece must be in perfect technical conditions.
- Insert the instruments carefully and without using force. Miniature instruments should be inserted no further than the end of the cylindrical part.
- Avoid canting or levering when grinding.
- Observe the recommended speeds indicated in the table below and on packages.
- Full speed should be reached outside the mouth of the patient. Technicians should run the handpiece at full speed before application to the product on which is being performed.

Hygiene recommendations (desinfection, cleaning, sterilization and storage)

- The instruments must always be disinfected, cleaned and sterilized before they are used on patients.
- Immediately place used instruments in a special anti-corrosive disinfectant bath for a short time otherwise their colour code may come off. Do not use too aggressive products (hydrochloric acid, hydrogen peroxide,...) they may corrode the instruments.
- Clean the instruments in an ultrasonic bath or under running water (preferably distilled water).
- After cleaning, immediately dry the instruments otherwise they can corrode.
- Before sterilization, place the instruments in stands and trays suitable for sterilization.
- Sterilize the instruments. Autoclave: 18 min. 134°C, 2 bars.
- After sterilization, keep the instruments in dust-free places. Instruments used for surgery should be kept in sealed containers.

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AZ & PARTNER AG